

# Start-Up Guide

## Meet your Pro Light Mask:

The Equilume patented blue-light technology, specifically developed for horses, strengthens circadian rhythms and improves the functioning of the body clock. Our Pro Light Mask provides 15 hours of blue light daily, mimicking optimal natural light exposure, helping your horse stay healthier from the inside out all year round. It features a fleece-lined headpiece, padded cup lining, and adjustable Velcro straps for a secure and comfortable fit.

## What Benefits to Expect:

- Enhances athletic performance
- Regulates mood and behavior
- Improves coat condition and shine
- Stimulates immunity and healing
- Supports reproductive health

## What's in the Box:

- Light Mask
- Wireless Charging Pad & USB-C Cable
- Smart Key

## Getting Started:

**Step 1:** Charge the Pro Light Mask using the wireless charger.

**Step 2:** Activate the Pro Light Mask at 8 am using the Smart Key.

**Step 3:** Fit the mask on your horse, adjusting for comfort and fit.

**Step 4:** Remove the Pro mask once every 7 days to charge.



## How to Charge:


**Set up the charger:** Plug the USB cable into a USB plug and connect it to an electrical outlet. Then, plug the USB-C end into the wireless charger's port and place the wireless charger in the cup, aligning the "Top" and "Bottom" labels. The charger will magnetically attach.

**Charging the cup:** The indicator light on the cup will flash red while charging (3-4 hours). When fully charged, the indicator light will turn solid red. Once charging is complete, remove the magnetic charger and wire from the cup.

## How to Activate:

Gently move the Equilume Smart Key across the cup surface. Activate at 8am on application day, before fitting the mask. Activation is only needed once. After 7 days of continuous use, the Light Mask must be charged within 24 hours to prevent battery depletion. Charging takes 3-4 hours.

Slide in  
the direction  
of the arrows

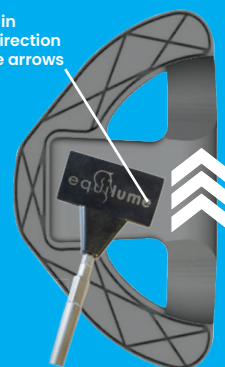

## How to Fit:

Open the Velcro below the jaw and slide the Light Mask over the horse's ears like a halter/bridle.

Using the three Velcro closures, adjust the fit and straighten the Light Mask. Ensure the eye openings are centered and the cup is positioned over half of the eye. Close the Velcro under the jaw, leaving space for two fingers.

If needed, fit a halter over the Light Mask, threading the head strap through the halter loops for a secure fit.

